

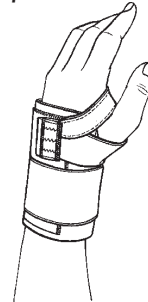


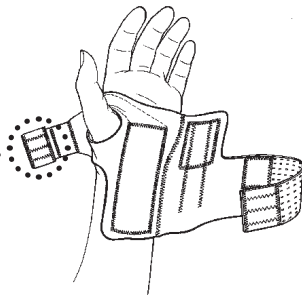

Motion Manager™ Wrist Support

Before using this product, please carefully read all instructions and warnings.

Application Instructions

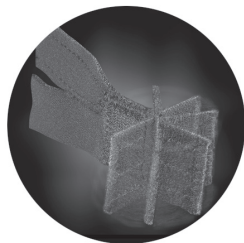

Motion Manager is reversible to fit both the left or right hand! The exclusive "Flip Tab" hook and loop fastener on the end of the thumb strap allows for both left/right and comfort adjustments.

Place thumb through hole in thumb strap. Position palmar panel (panel attached adjacent to thumb strap) in palm of hand. Be certain that the molded plastic stay in the palmar panel is in the outside pocket (away from the palm) and the contour of the molded stay matches the palm.

Wrap the wrist strap around the wrist and secure by engaging the loop panel at the end of the wrist strap to the hook panel at the base of the wrist strap. Adjust engagement to achieve desired compression.

If necessary, flip up the end of the flip tab hook-and-loop fastener at the end of the thumb strap to secure it to the hook panel located on the dorsal panel (back of hand). Fold the loose end of the tab back on itself to finish. Adjust engagement to achieve desired comfort.

Note: You will find an additional plastic stay for the dorsal panel included with this Motion Manager. Use this stay as a replacement if more resistance to extension is desired.

Washing Instructions: Engage all closure straps before washing. Machine wash, warm water, gentle cycle, no bleach, air dry

Warning: If you experience any increased pain, swelling, or any unusual reactions while using this product, immediately consult your medical professional. This product is for single patient use only. There is no guarantee that use of this product will prevent injury.

Warranty: Products manufactured by Med Spec have a limited warranty against defects in materials or workmanship for a period of six months from date of shipment. Please contact the business that supplied you with this product if you wish to return it.

Medical Specialties, Inc.
4600-K Lebanon Rd.
Charlotte, NC 28227 USA
p: 800-334-4143 f: 800-694-9060
p: 704-694-2434 f: 704-694-9060
e-mail: request@medspec.com
www.medspec.com

Motion Manager is a trademark of
Medical Specialties, Inc.
Med Spec is a registered trademark of
Medical Specialties, Inc.