





# GelFlex<sup>®</sup> Wrist

Before using this product, please carefully read all instructions and warnings.

## Application Instructions

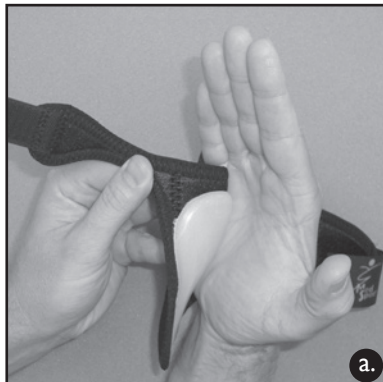

Detach all hook & loop fasteners and remove plastic film from gel pad. Locate gel pad so it matches the curvature of the palm.

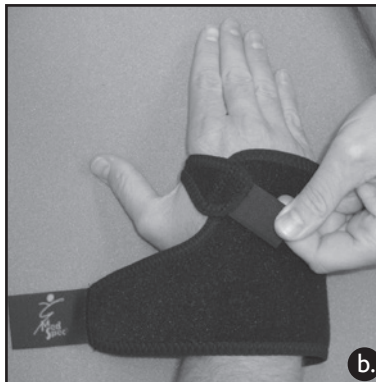

Pull thumb strap and fasten to wrist support.



Pull wrist strap and fasten to wrist support.

**Warning:** If you experience any increased pain, swelling, or any unusual reactions while using this product, immediately consult your medical professional. This product is for single patient use only. There is no guarantee that use of this product will prevent injury.

**Washing Instructions:** hand wash, warm water, mild detergent, no bleach, air dry

**Note:** Plastic stay may be removed if advised by medical provider.

**Warranty:** Products manufactured by Med Spec have a limited warranty against defects in materials or workmanship for a period of six months from date of shipment. Please contact the business that supplied you with this product if you wish to return it.



MDSS GmbH  
Schiffgraben 41  
30175 Hannover, Germany



Medical Specialties, Inc.  
4600-K Lebanon Rd.  
Charlotte, NC 28227  
p: 800-334-4143 f: 800-694-9060  
p: 704-694-2434 f: 704-694-9060  
e-mail: request@medspec.com  
www.medspec.com