

Tee Pee™ Thumb Protector

Instructions For Use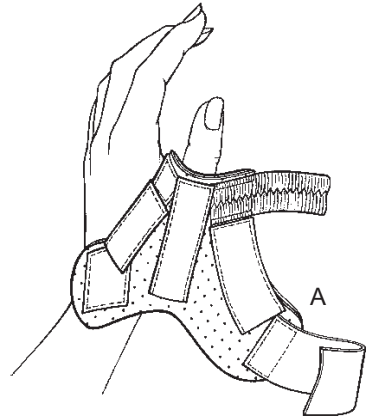

A. If necessary, gently bend the main/central stay (a.) to achieve the desired position of the thumb. Position the product so that the main/central stay lays along the extensor surface (back) of the injured thumb, as shown. **Gently bend the palmar stay** inward to create a contoured, custom fit on the palm side of the thumb.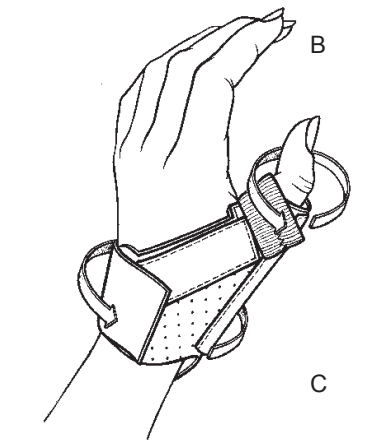

B. Wrap the thumb strap (b.) around the thumb and secure by engaging the hook panel. Adjust engagement to achieve desired comfort.

Medical Specialties, Inc.
4600-K Lebanon Rd.
Charlotte, NC 28227 USA
p: 800-334-4143 f: 800-694-9060
p: 704-694-2434 f: 704-694-9060
e-mail: request@medspec.com
www.medspec.com

Washing Instructions: Hand wash in warm water with mild detergent and no bleach. Rinse thoroughly. Air dry.

Warranty: Products manufactured by Med Spec have a limited warranty against defects in materials or workmanship for a period of six months from date of shipment. Please contact the business that supplied you with this product if you wish to return it.

EC REP
MEDSS GmbH
Schiffgraben 41
30175 Hannover, Germany

English

Before using this product, please carefully read all instructions and warnings.

Warning: If you experience any increased pain, swelling, or any unusual reactions while using this product, immediately consult your medical professional. This product is for single patient use only. There is no guarantee that use of this product will prevent injury.

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