



ASO[®] Vortex[®]

Application Instructions

Before using this product, please carefully read all instructions and warnings.



Place your foot with sock inside the upper body member of the **ASO[®] Vortex[®]** Ankle Stabilizer. Feed lace through remaining eyelets, tighten and tie. **Make sure lace is tight between all eyelets and tied securely to ensure adequate support.**



Bring one stabilizing strap forward across the top of the foot.



Continue wrapping first stabilizing strap under the heel and attach the end of strap to the side of the boot. Repeat steps above for the other stabilizing strap.



Place foot flat on the floor. Use the finger loops at the end of both straps to free them from the sides of the upper body member. **IMPORTANT: PULL STRAPS UP FIRMLY. Stabilizing straps must be tight for proper support.** Reattach straps to the upper body member. Repeat if needed.



Wrap elastic cuff closure over tied laces and secured straps. For optimum performance and support, the elastic cuff should be stretched to where the **Vortex** logo is facing forward.

Washing Instructions: The ASO Ankle Stabilizer may be hand or machine washed on gentle cycle with warm water and mild detergent. Do not use bleach. The ASO must be air dried.

Warning: If you experience any increased pain, swelling, or any unusual reactions while using this product, immediately consult your medical professional. This product is for single patient use only. There is no guarantee that use of this product will prevent injury.

Warranty: Products manufactured by Med Spec have a limited warranty against defects in materials or workmanship for a period of six months from date of shipment. Please contact the business that supplied you with this product if you wish to return it.



Medical Specialties, Inc.
4600-K Lebanon Rd.
Charlotte, NC 28227 USA
p: 800-334-4143 f: 800-694-9060
p: 704-694-2434 f: 704-694-9060
e-mail: request@medspec.com
www.medspec.com



MDSS GmbH
Schiffgraben 41
30175 Hannover, Germany