# Gripper™ Hinged Knee

## Instructions For Use

A. Disengage all closure straps. Position knee brace so opening closes on the anterior (front) side of the leg and the patella cut-out is centered around the patella (knee cap).

B. Fasten inner (wide) flaps above and below the knee. Make sure patella is centered on patella cut-out and adjust position of knee brace if necessary.

C. Fasten outer (nylon) straps above and below the knee.

**Washing Instructions:** Engage all closure straps before machine washing. Hand or machine wash on gentle cycle in warm water with mild detergent and no bleach. Rinse thoroughly. Air dry.

**Warranty:** Products manufactured by Med Spec have a limited warranty against defects in materials or workmanship for a period of six months from date of shipment. Please contact the business that supplied you with this product if you wish to return it.

### English

Before using this product, please carefully read all instructions and warnings.

**Warning:** If you experience any increased pain, swelling, or any unusual reactions while using this product, immediately consult your medical professional. This product is for single patient use only. There is no guarantee that use of this product will prevent injury.

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Gripper is a trademark of Medical Specialties, Inc.
Med Spec is a registered trademark of Medical Specialties, Inc.
Medical Specialties, Inc.
MDSS GmbH