
Gripper™ ROM Hinged Knee


Instructions For Use

A. Disengage all hook & loop closure straps, including separation of all straps from side of knee brace. The 16" version of the Gripper ROM has 4 closure straps and the 12" version has 2 closure straps.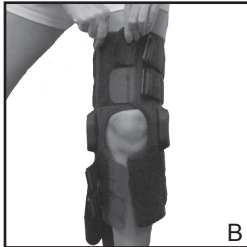

B. Position knee brace with wide inner flaps facing anterior (front) of the leg. Align brace by centering patella opening around patella (knee cap), then fasten inner flaps.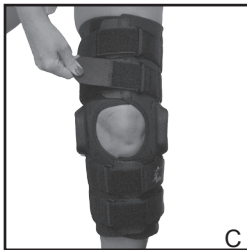

C. Before applying straps, adjust knee brace if necessary to ensure patella opening is centered around patella, and then fasten outer (nylon) straps above and below the knee.

D. Consult with your medical professional before changing position of flexion and extension stops. To adjust position of flexion and extension stops, disengage hook & loop closure underneath padded hinge cover and then flip open plastic hinge cover.

Washing Instructions: Hand wash in warm water with mild detergent and no bleach. Rinse thoroughly. Air dry.

Warranty: Products manufactured by Med Spec have a limited warranty against defects in materials or workmanship for a period of six months from date of shipment. Please contact the business that supplied you with this product if you wish to return it.

English

Before using this product, please carefully read all instructions and warnings.

Warning: If you experience any increased pain, swelling, or any unusual reactions while using this product, immediately consult your medical professional. This product is for single patient use only. There is no guarantee that use of this product will prevent injury.

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Gripper is a trademark of Medical Specialties, Inc.
Med Spec is a registered trademark of Medical Specialties, Inc.
Medical Specialties, Inc.
MDSS GmbH