




# Archimed<sup>®</sup> Universal Spinal Brace

Before using this product, please carefully read all instructions and warnings.

## *Application Instructions*

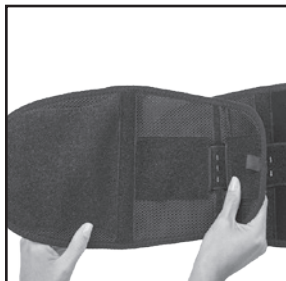

Disengage the hook & loop fasteners between the outer panels at each end and slide the outer panel to adjust for proper fit. Reattach the hook & loop fasteners at each end.



Position Archimed with posterior (back) panel centered on your lower spine.



Overlap anterior (front) panels and secure hook and loop closure.



Detach straps connected to laces and pull to desired level of tension.



Fasten straps to anterior (front) panels.

Note: Eyelet-lace pods over posterior (back) panel may be adjusted to provide a better fit. If required, you may purchase an Extension panel (P/N 163390) to fit larger patients up to 72".

**Washing instructions:** Engage all closure straps before washing. Hand wash, warm water, no bleach, rinse, air dry

**Warning:** If you experience any increased pain, swelling, or any unusual reactions while using this product, immediately consult your medical professional. This product is for single patient use only. There is no guarantee that use of this product will prevent injury.

**Warranty:** Products manufactured by Med Spec have a limited warranty against defects in materials or workmanship for a period of six months from date of shipment. Please contact the business that supplied you with this product if you wish to return it.

Medical Specialties, Inc.  
4600-K Lebanon Rd.  
Charlotte, NC 28227 USA  
p: 800-334-4143 f: 800-694-9060  
p: 704-694-2434 f: 704-694-9060  
e-mail: request@medspec.com  
www.medspec.com