Med Spec® Cubital Tunnel Brace

Instructions For Use

A. Loosen all straps and slide brace over affected arm with Med Spec logo towards shoulder.

B. Position brace on inside of arm so the bend in the stay is centered just above the elbow crease and secure middle strap first.

C. Tighten and secure distal (forearm) strap and then proximal (bicep) strap. All three straps should be tight enough to hold the brace in position, but not so tight as to restrict circulation.

Washing Instructions: Hand wash in warm water with mild detergent and no bleach. Rinse thoroughly. Air dry.

Warranty: Products manufactured by Med Spec have a limited warranty against defects in materials or workmanship for a period of six months from date of shipment. Please contact the business that supplied you with this product if you wish to return it.

English

Before using this product, please carefully read all instructions and warnings.

Warning: If you experience any increased pain, swelling, or any unusual reactions while using this product, immediately consult your medical professional. This product is for single patient use only. There is no guarantee that use of this product will prevent injury.

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Medical Specialties, Inc.
MDSS GmbH