



Med Spec[®] Padded Arm Sling

Before using this product, please carefully read all instructions and warnings.



Application instructions:

1. Detach hook and loop closure on padded shoulder strap and reattach so strap is at its maximum length.
2. Place shoulder strap overhead so it rest on back of neck.
3. Slide hand of affected shoulder inside envelope below plastic buckles. Then slide elbow fully into corner of sling.
4. Tighten shoulder strap so that forearm is resting with hand slightly above elbow. Insert thumb in thumb strap on inside of envelope.

Note: This padded arm sling is for single patient use only.

Washing instructions: Engage all closure straps before washing. Hand wash, warm water, no bleach, rinse, air dry

Warning: If you experience any increased pain, swelling, or any unusual reactions while using this product, immediately consult your medical professional. This product is for single patient use only. There is no guarantee that use of this product will prevent injury.

Warranty: Products manufactured by Med Spec have a limited warranty against defects in materials or workmanship for a period of six months from date of shipment. Please contact the business that supplied you with this product if you wish to return it.

Medical Specialties, Inc.
4600-K Lebanon Rd. Charlotte, NC 28227
p: 800-334-4143 f: 800-694-9060
p: 704-694-2434 f: 704-694-9060
e-mail: request@medspec.com
www.medspec.com