Universal Hemi-Sling

Instructions For Use

A. Remove connecting strap from Hemi Sling. Hold by metal “O” ring and separate shoulder straps. Lifting “O” ring overhead, place unaffected arm through wrist cuff, allowing “O” ring to lie on back of neck.

B. Slide elbow of impaired arm into elbow cuff until it rests against strap closure end.

C. Place unaffected arm through wrist cuff and lift impaired hand into wrist cuff.

D. Secure connecting strap to front of both cuffs. Shoulder strap length may be adjusted by sliding buckle.

Medical Specialties, Inc.
4600-K Lebanon Rd.

Charlotte, NC 28227 USA
p: 800-334-4143 f: 800-694-9060
p: 704-694-2434 f: 704-694-9060
e-mail: request@medspec.com
www.medspec.com

Washing Instructions: Hand wash in warm water with mild detergent and no bleach. Rinse thoroughly. Air dry.

Warranty: Products manufactured by Med Spec have a limited warranty against defects in materials or workmanship for a period of six months from date of shipment. Please contact the business that supplied you with this product if you wish to return it.

English

Before using this product, please carefully read all instructions and warnings.

Warning: If you experience any increased pain, swelling, or any unusual reactions while using this product, immediately consult your medical professional. This product is for single patient use only. There is no guarantee that use of this product will prevent injury.

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MDSS GmbH