



Outrigger® II Shoulder Immobilizer

Before using this product please carefully read all instructions and warnings.

Application Instructions



Separate pillow from sling and disconnect both plastic buckles. Slide forearm and elbow of injured shoulder into sling. Adjust length of shoulder strap and secure plastic buckle to sling.



Attach pillow to sling. Adjust length of waist strap and secure plastic buckle to pillow.



Disengage tab on shoulder strap pad and position pad around base of neck.



Secure thumb strap to sling.

Note: Subsequent application requires only snapping the plastic buckles for waist & shoulder straps and attaching thumb strap. Pillow and sling are bilateral (fit left or right shoulder). Outrigger II may be worn without pillow if prescribed by your medical provider.

Warning: If you experience any increased pain, swelling, or any unusual reactions while using this product, immediately consult your medical professional. This product is for single patient use only. There is no guarantee that use of this product will prevent injury.

Washing instructions: Engage all closure straps before washing. Hand wash, warm water, no bleach, rinse, air dry

Warranty: Products manufactured by Med Spec have a limited warranty against defects in materials or workmanship for a period of six months from date of shipment. Please contact the business that supplied you with this product if you wish to return it.



Medical Specialties, Inc.
4600-K Lebanon Rd.
Charlotte, NC 28227
p: 800-334-4143 f: 800-694-9060
p: 704-694-2434 f: 704-694-9060
e-mail: request@medspec.com
www.medspec.com