SI Belt

Instructions For Use

English

Before using this product, please carefully read all instructions and warnings.

Warning: If you experience any increased pain, swelling, or any unusual reactions while using this product, immediately consult your medical professional. This product is for single patient use only. There is no guarantee that use of this product will prevent injury.

A. Position SI Belt with sacral pad centered on lower back over waistline.

B. Wrap SI Belt around waist with overlapping ends of inner elastic belt across abdomen and secure hook and loop closure.

C. Insert outer nylon strap through plastic buckle, then pull tight for firm support and secure hook and loop closure.

Washing Instructions: Hand wash in warm water with mild detergent and no bleach. Rinse thoroughly. Air dry.

Warranty: Products manufactured by Med Spec have a limited warranty against defects in materials or workmanship for a period of six months from date of shipment. Please contact the business that supplied you with this product if you wish to return it.

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Medical Specialties, Inc.
MDSS GmbH