
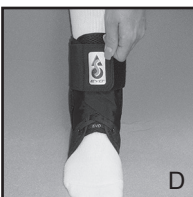
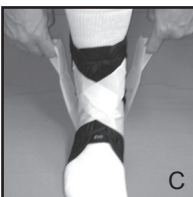
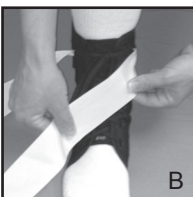
# EVO<sup>®</sup>, EVO<sup>®</sup> w/Hinge & EVO<sup>®</sup> Quatro<sup>®</sup> Ankle Stabilizer Instructions For Use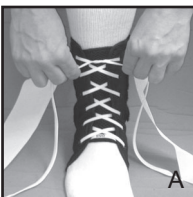

## English

Before using this product, please carefully read all instructions and warnings.

**Warning:** If you experience any increased pain, swelling, or any unusual reactions while using this product, immediately consult your medical professional. This product is for single patient use only. There is no guarantee that use of this product will prevent injury.

A. Place your foot with sock inside the boot of the **EVO<sup>®</sup>, EVO<sup>®</sup> w/ Hinge** or **EVO<sup>®</sup> Quatro<sup>®</sup>** Ankle Stabilizer. Feed lace through remaining eyelets, tighten and tie. **Make sure lace is tight between all eyelets and tied securely to ensure adequate support.**

B. Bring one stabilizing strap forward and across the top of the foot. Continue wrapping first stabilizing strap under the heel and attach the end of strap to the side of the boot. Repeat steps above for the other stabilizing strap.

C. Place foot flat on the floor. Use the finger loops at the end of both straps to free them from the sides of the boot. **IMPORTANT: PULL STRAPS UP FIRMLY.** Stabilizing straps must be tight for proper support. Reattach straps to boot. Repeat if needed.

D. Wrap elastic cuff closure over tied laces and secured straps. For optimum performance and support, the elastic cuff should be stretched to a point where the **EVO** or **Quatro** logo is facing forward.

E. **EVO<sup>®</sup> w/ Hinge** Ankle Stabilizer may also be worn without plastic hinge as advised by your medical provider. To remove plastic hinge, disengage all straps, then disengage hook & loop fasteners on plastic hinge from both sides of boot.

**Washing Instructions:** Engage all closure straps before machine washing. Hand or machine wash on gentle cycle in warm water with mild detergent and no bleach. Rinse thoroughly. Air dry.

**Warranty:** Products manufactured by Med Spec have a limited warranty against defects in materials or workmanship for a period of six months from date of shipment. Please contact the business that supplied you with this product if you wish to return it.

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Medical Specialties, Inc.
MDSS GmbH