Supinator™ PTT Stabilizer

Instructions For Use

A. Detach all hook and loop fasteners. Place your foot with sock inside the boot portion of the *Supinator™* PTT Stabilizer. Pull both lace closures in opposite directions and fasten to sides of boot. **Make sure laces are tight to ensure adequate support.**

B. Position pad under arch in desired location. Note: arch strap should pass through guide on pad.

C. Bring arch strap across the top of the foot, pull strap up firmly to lift the arch and attach end of strap near **upper** edge of boot.

D. Bring heel strap forward and across the top of the foot and continue under the heel. Then, pull strap up firmly and fasten the end of strap to the side of the boot.

E. Wrap elastic cuff closure over laces and straps and fasten the elastic cuff so the **Med Spec** logo is facing forward.

Washing Instructions: Engage all closure straps before machine washing. Hand or machine wash on gentle cycle in warm water with mild detergent and no bleach. Rinse thoroughly. Air dry.

Warranty: Products manufactured by Med Spec have a limited warranty against defects in materials or workmanship for a period of six months from date of shipment. Please contact the business that supplied you with this product if you wish to return it.

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Supinator™ is a trademark of Medical Specialties, Inc.
Medical Specialties, Inc.
MDSS GmbH