




# DynaTrack™ With Metal Hinges

Before using this product, please carefully read all instructions and warnings.

## Application Instructions



Disengage all straps. Position knee brace so opening closes on the anterior (front) side of the leg. Fasten (short) inner straps above and below the knee.

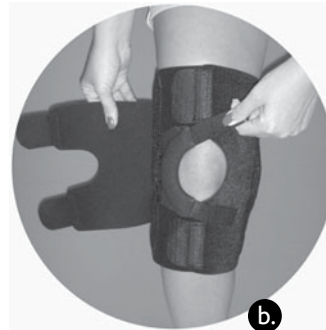

Pull and fasten buttress straps to apply desired pressure against patella (knee cap). Make sure buttress is properly positioned against side of patella.



Pull and fasten (long) outer straps to apply desired pressure against patella.



Medical Specialties, Inc.  
4600-K Lebanon Rd.  
Charlotte, NC 28227  
p: 800-334-4143 f: 800-694-9060  
p: 704-694-2434 f: 704-694-9060  
e-mail: request@medspec.com  
www.medspec.com

**Washing instructions:** Engage all closure straps before washing. Machine wash on gentle cycle with mild detergent and no bleach. Air dry.

**Warranty:** Products manufactured by Med Spec have a limited warranty against defects in materials or workmanship for a period of six months from date of shipment. Please contact the business that supplied you with this product if you wish to return it.