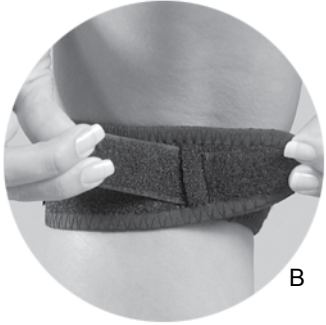
Patellavator®

Knee Orthosis

Instructions For Use

A

B

C

Medical Specialties, Inc.
4600-K Lebanon Rd.
Charlotte, NC 28227 USA
p: 800-334-4143 f: 800-694-9060
p: 704-694-2434 f: 704-694-9060
e-mail: request@medspec.com
www.medspec.com

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English

Before using this product, please carefully read all instructions and warnings.

Warning: If you experience any increased pain, swelling, or any unusual reactions while using this product, immediately consult your medical professional. This product is for single patient use only. There is no guarantee that use of this product will prevent injury.

A. Detach outer laces from base strap. Place **Patellavator** knee orthosis with Med Spec logo right side up, just below patella (knee cap).

B. Insert angled strap through slot behind knee to create an interlocking closure. **Pull the strap completely through (no hook fastener material should be next to skin). If uncomfortable to pull too tight, select larger size.**

C. Pull external laces outward until tight and secure to base strap.

Washing Instructions: Engage all closure straps before machine washing. Hand or machine wash on gentle cycle in warm water with mild detergent and no bleach. Rinse thoroughly. Air dry.

Warranty: Products manufactured by Med Spec have a limited warranty against defects in materials or workmanship for a period of six months from date of shipment. Please contact the business that supplied you with this product if you wish to return it.

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