



Tripod™ Lacer II Wrist Support

Before using this product, please carefully read all instructions and warnings.

Application Instructions



A. Detach all hook & loop fasteners and slide hand under stockinette and move side to side to equalize tension along each individual lace then fasten to wrist support.



B. Pull strap/lace closure tight and move side to side to equalize tension along each individual lace then fasten to wrist support.



C. Fasten thumb strap so that it is aligned with hook and loop fasteners.

Warning: If you experience any increased pain, swelling, or any unusual reactions while using this product, immediately consult your medical professional. This product is for single patient use only. There is no guarantee that use of this product will prevent injury.

Washing instructions: Engage all closure straps before machine washing. Hand or machine wash on gentle cycle in warm water with mild detergent and no bleach. Rinse thoroughly. Air dry.

Warranty: Products manufactured by Med Spec have a limited warranty against defects in materials or workmanship for a period of six months from date of shipment. Please contact the business that supplied you with this product if you wish to return it.

Medical Specialties Inc.
4600-K Lebanon Rd.
Charlotte, NC 28227 USA
p: 800-334-4143 f: 800-694-9060
p: 704-694-2434 f: 704-694-9060
e-mail: request@medspec.com
www.medspec.com

