





# Archimed™ Universal Extension Panel

Before using this product, please carefully read all instructions and warnings.


*Application Instructions*

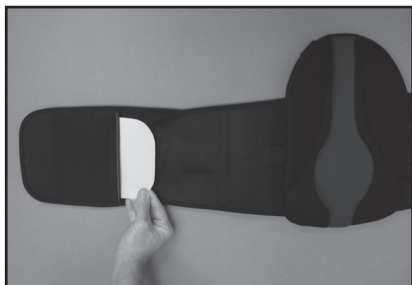

Open pocket on inside of brace and remove plastic plate from outer end of panel.

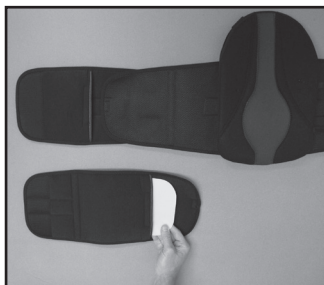

Insert plastic plate inside extension panel pocket with the edges of the plastic plate curved upward.

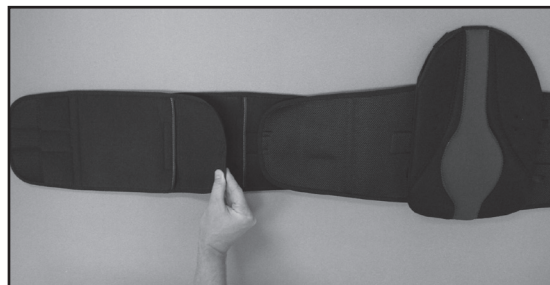

Attach extension panel to inside of end panel.

**Washing instructions:** Engage all closure straps before washing. Hand wash, warm water, no bleach, rinse, air dry

**Warning:** If you experience any increased pain, swelling, or any unusual reactions while using this product, immediately consult your medical professional. This product is for single patient use only. There is no guarantee that use of this product will prevent injury.

**Warranty:** Products manufactured by Med Spec have a limited warranty against defects in materials or workmanship for a period of six months from date of shipment. Please contact the business that supplied you with this product if you wish to return it.



Medical Specialties Inc.  
4600-K Lebanon Rd.  
Charlotte, NC 28227  
p: 800-334-4143 f: 800-694-9060  
p: 704-694-2434 f: 704-694-9060  
e-mail: request@medspec.com  
www.medspec.com

