



Back-n-Black™ with Dual Panels

Before using this product, please carefully read all instructions and warnings.



Application Instructions

1. Detach anterior (front) panel from back support.
2. Position Back-n-Black with posterior thermoplastic insert centered on your lower back.
3. Wrap back support around your waist and overlap ends of elastic belt over abdomen and secure hook and loop closure.
4. Position anterior panel over abdomen and reattach to elastic belt.
5. Insert nylon strap through plastic buckle, pull tight for firm support and secure hook and loop closure.

Washing instructions: Engage all closure straps before washing. Hand wash, warm water, no bleach, rinse, air dry

Warning: If you experience any increased pain, swelling, or any unusual reactions while using this product, immediately consult your medical professional. This product is for single patient use only. There is no guarantee that use of this product will prevent injury.

Warranty: Products manufactured by Med Spec have a limited warranty against defects in materials or workmanship for a period of six months from date of shipment. Please contact the business that supplied you with this product if you wish to return it.

 Medical Specialties Inc.
4600-K Lebanon Rd.
Charlotte, NC 28227
p: 800-334-4143 f: 800-694-9060
p: 704-694-2434 f: 704-694-9060
e-mail: request@medspec.com
www.medspec.com

