




Universal Foam Sling & Swathe Shoulder Immobilizer


Before using this product, please carefully read all instructions and warnings.

Application Instructions

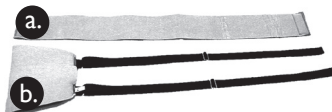

The Universal Foam Sling & Swathe shoulder immobilizer consists of two components:

- a. foam swathe
- b. foam sling

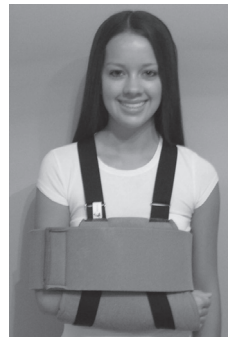

Warning: If you experience any increased pain, swelling, or any unusual reactions while using this product, immediately consult your medical professional. This product is for single patient use only. There is no guarantee that use of this product will prevent injury.

Step 1: Slide arm into gray foam sling. Position each black strap over the left and right shoulder and criss-cross the straps over the back. Pull the end of each strap under the arm and fasten the strap to the front of the sling. To ensure optimum fit, adjust the length of each strap by sliding the metal buckle.

Step 2: Attach one end of the foam swathe to the front of the foam sling. Wrap the foam swathe around the outside of the injured arm and next to the body, inside the healthy arm. Fasten the free end to the front of the foam swathe.



Medical Specialties Inc.
4600-K Lebanon Rd.
Charlotte, NC 28227
p: 800-334-4143 f: 800-694-9060
p: 704-694-2434 f: 704-694-9060
e-mail: request@medspec.com
www.medspec.com

Washing Instructions: Engage all closure straps before washing. Machine wash, warm water, gentle cycle, no bleach, air dry

Warranty: Products manufactured by Med Spec have a limited warranty against defects in materials or workmanship for a period of six months from date of shipment. Please contact the business that supplied you with this product if you wish to return it.

