



Gripper™ 3 Hinged Knee

Before using this product, please carefully read all instructions and warnings.

Application Instructions



A. Disengage all closure straps. Position knee brace so opening closes on the anterior (front) side of the leg and the patella cut-out is centered around the patella (kneecap). Condylar pads may be repositioned medially or laterally by a medical professional to provide a custom fit.



B. Fasten inner (wide) flaps above and below the knee. Make sure patella is centered on patella cut-out and adjust position of knee brace if necessary.



C. Fasten outer straps above and below the knee. If an outer strap is too long, your medical professional may remove the Y shaped “alligator” hook fastener tab from the strap, cut the strap in small increments to the desired length, and then reattach the “alligator” fastener to the end of the strap.

Washing instructions: Engage all closure straps before machine washing. Hand or machine wash on gentle cycle in warm water with mild detergent and no bleach. Rinse thoroughly. Air dry.

Warranty: Products manufactured by Med Spec have a limited warranty against defects in materials or workmanship for a period of six months from date of shipment. Please contact the business that supplied you with this product if you wish to return it.

Medical Specialties Inc.
4600-K Lebanon Rd.
Charlotte, NC 28227 USA
p: 800-334-4143 f: 800-694-9060
p: 704-694-2434 f: 704-694-9060
e-mail: request@medspec.com
www.medspec.com

