



# Boxer Splint

Before using this product, please carefully read all instructions and warnings.

## Application Instructions




Bend internal aluminum stays to the degree of flexion prescribed by your medical professional.



Detach all straps and insert fingers and hand. Make sure the 5th (pinky) finger slides inside inner sleeve.



Wrap elastic strap around 4th and 5th (ring and pinky) fingers and secure.

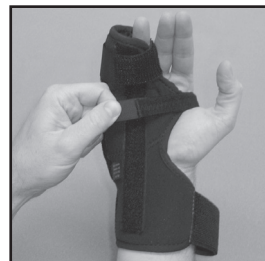

Tension strap connected to buckle and secure in palm.



Wrap elastic strap around wrist and secure.

<u>size</u>	<u>wrist circ.</u>
x-small	4" - 5"
small	5" - 6"
medium	6" - 7"
large	7" - 8 $\frac{1}{4}$ "
x-large	8 $\frac{1}{4}$ " - 9 $\frac{1}{2}$ "

**Washing Instructions:** Engage all closure straps before machine washing. Hand or machine wash on gentle cycle in warm water with mild detergent and no bleach. Rinse thoroughly. Air dry.

**Warning:** If you experience any increased pain, swelling, or any unusual reactions while using this product, immediately consult your medical professional. This product is for single patient use only. There is no guarantee that use of this product will prevent injury.

**Warranty:** Products manufactured by Med Spec have a limited warranty against defects in materials or workmanship for a period of six months from date of shipment. Please contact the business that supplied you with this product if you wish to return it.

Medical Specialties Inc.

4600-K Lebanon Rd. Charlotte, NC 28227 USA

p: 800-334-4143 f: 800-694-9060

e-mail: request@medspec.com

www.medspec.com

U.S. Patent No. 8,246,560

