



Back-n-Black™ and Back-n-Black™ with Thermo Pocket

Before using this product, please carefully read all instructions and warnings.



FRONT



*BACK

Application Instructions

1. Position back support centered on lower back.
2. Wrap back support around waist overlapping ends of elastic belt across abdomen and secure hook and loop closure.
3. Insert nylon strap through plastic buckle, pull tight for firm support and secure hook and loop closure.

* Back-n-Black with Thermo Pocket showing thermoplastic insert properly centered on lower back. Thermoplastic insert is partially removed for demonstration purposes and is sold separately (P/N 163000).

Washing instructions: Hand wash in warm water with mild detergent and no bleach. Rinse thoroughly. Air dry.

Warning: If you experience any increased pain, swelling, or any unusual reactions while using this product, immediately consult your medical professional. This product is for single patient use only. There is no guarantee that use of this product will prevent injury.

Warranty: Products manufactured by Med Spec have a limited warranty against defects in materials or workmanship for a period of six months from date of shipment. Please contact the business that supplied you with this product if you wish to return it.



Medical Specialties Inc. 4600-K Lebanon Rd. Charlotte, NC 28227 USA
p: 800-334-4143 f: 800-694-9060 p: 704-694-2434 f: 704-694-9060
e-mail: request@medspec.com www.medspec.com
Back-n-Black™ is a trademark of Medical Specialties Inc.