ASO® Speed Lacer™ II

Ankle Stabilizer

Instructions For Use

A

B

C

D

E

Medical Specialties, Inc.
4600-K Lebanon Rd.
Charlotte, NC 28227 USA
p: 800-334-4143 f: 800-694-9060
p: 704-694-2434 f: 704-694-9060
e-mail: request@medspec.com
www.medspec.com

EC

REP

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ASO® is a registered trademark of Medical Specialties, Inc.
Speed Lacer™ is a trademark of Medical Specialties, Inc.

English

Before using this product, please carefully read all instructions and warnings.

Warning: If you experience any increased pain, swelling, or any unusual reactions while using this product, immediately consult your medical professional. This product is for single patient use only. There is no guarantee that use of this product will prevent injury.

A. Detach all hook and loop fasteners. Place your foot with sock inside the boot portion of the **ASO® Speed Lacer™ II** Ankle Stabilizer. Pull lower two lace tab closures in opposite directions and fasten to sides of boot. Pull remaining upper lace tab closure and fasten to side of boot. **Make sure laces are tight to ensure adequate support.**

B. Bring one stabilizing strap forward and across the top of the foot.

C. Continue wrapping first stabilizing strap under the heel and attach the end of strap to the side of the boot. Repeat steps above for the other stabilizing strap.

D. Place foot flat on the floor. Use the finger loops at the end of both straps to free them from the sides of the boot. **IMPORTANT: PULL STRAPS UP FIRMLY. Stabilizing straps must be tight for proper support.** Reattach straps to boot. Retighten straps while wearing for optimum support.

E. Wrap elastic cuff closure over laces and secure in rear.

Washing Instructions: Engage all closure straps before machine washing. Hand or machine wash on gentle cycle in warm water with mild detergent and no bleach. Rinse thoroughly. Air dry.

Warranty: Products manufactured by Med Spec have a limited warranty against defects in materials or workmanship for a period of six months from date of shipment. Please contact the business that supplied you with this product if you wish to return it.

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MDSS GmbH Schiffgraben 41 30175 Hannover, Germany