EpiFoam™
Tennis Elbow
Support

Instructions For Use

A. Feed strap through plastic buckle with Velcro hook facing up. Make sure the foam pad is attached directly underneath plastic buckle.

B. Slide the EpiFoam up your forearm to the bend in your elbow so that the pad underneath the plastic buckle will apply pressure to the lateral (outside) muscles and tendons. Do not pull strap at an angle. Pull end of strap straight over inner strap.

C. Pull strap tight until you feel tension, but do not over tighten. Wrap end of strap around forearm to secure brace.

Washing Instructions: Hand wash in warm water with mild detergent and no bleach. Rinse thoroughly. Air dry.

Warranty: Products manufactured by Med Spec have a limited warranty against defects in materials or workmanship for a period of six months from date of shipment. Please contact the business that supplied you with this product if you wish to return it.

Medical Specialties, Inc.
4600-K Lebanon Rd.
Charlotte, NC 28227 USA
p: 800-334-4143 f: 800-694-9060
p: 704-694-2434 f: 704-694-9060
e-mail: request@medspec.com
www.medspec.com

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EpiFoam is a trademark of Medical Specialties Inc.

English

Before using this product, please carefully read all instructions and warnings.

Warning: If you experience any increased pain, swelling, or any unusual reactions while using this product, immediately consult your medical professional. This product is for single patient use only. There is no guarantee that use of this product will prevent injury.

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Medical Specialties, Inc. 4600-K Lebanon Rd. Charlotte, NC 28227 USA p: 800-334-4143 f: 800-694-9060 p: 704-694-2434 f: 704-694-9060 e-mail: request@medspec.com www.medspec.com MDSS GmbH Schiffgraben 41 30175 Hannover, Germany EpiFoam is a trademark of Medical Specialties Inc.