



Boxer Splint II

Before using this product, please carefully read all instructions and warnings.

Application Instructions

BOXER'S FRACTURE INSTRUCTIONS



Detach all straps and insert pinky finger (fifth digit) in the inner sleeve. Secure the pinky and ring fingers (fourth and fifth digits) with the shortest end (distal) strap. Trim inner finger sleeve if desired.



Wrap the middle strap around the palm and secure.



Wrap the widest (proximal) strap around the wrist and secure.



Application complete.

RADIAL GUTTER INSTRUCTIONS



Detach all straps and insert index finger (second digit) in the inner sleeve. Trim inner finger sleeve if desired.



Secure the index and middle fingers (second and third digits) with the shortest end (distal) strap.



Wrap the middle strap around the palm and secure.



Wrap the widest (proximal) strap around the wrist and secure.



Application complete.

METACARPAL FRACTURE INSTRUCTIONS

Detach all straps and insert ring finger (fourth digit) in the inner sleeve. Trim inner finger sleeve if desired.

Secure the middle and ring fingers (third and fourth digits) with the shortest end (distal) strap.

Wrap the middle strap around the palm and secure.

Wrap the widest (proximal) strap around the wrist and secure.



Application complete.

Washing Instructions: Engage all closure straps before machine washing. Hand or machine wash on gentle cycle in warm water with mild detergent and no bleach. Rinse thoroughly. Air dry.

Warning: If you experience any increased pain, swelling, or any unusual reactions while using this product, immediately consult your medical professional. This product is for single patient use only. There is no guarantee that use of this product will prevent injury.

Warranty: Products manufactured by Med Spec have a limited warranty against defects in materials or workmanship for a period of six months from date of shipment. Please contact the business that supplied you with this product if you wish to return it.

