

TriFib™ Wrist Support


Instructions For Use

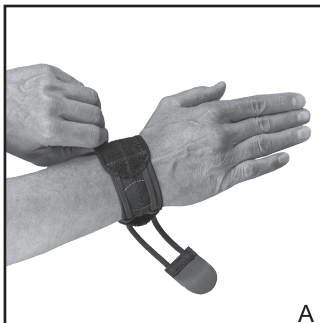A. Detach lace closure strap and slide hand into wrist support. Position circular teal stitching over the protruding bone on the little finger side of wrist (ulnar styloid process).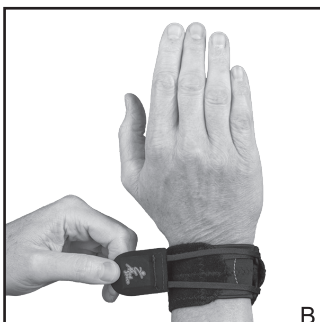

B. Pull lace closure strap tight and fasten to wrist support.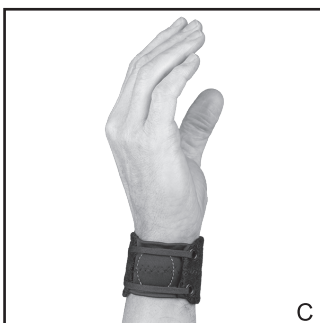

C. Reposition circular teal stitching and retighten lace closure strap as needed for maximum support.

Washing Instructions: Engage all closure straps before machine washing. Hand or machine wash on gentle cycle in warm water with mild detergent and no bleach. Rinse thoroughly. Air dry.

Warranty: Products manufactured by Med Spec have a limited warranty against defects in materials or workmanship for a period of six months from date of shipment. Please contact the business that supplied you with this product if you wish to return it.

English

Before using this product, please carefully read all instructions and warnings.

Warning: If you experience any increased pain, swelling, or any unusual reactions while using this product, immediately consult your medical professional. This product is for single patient use only. There is no guarantee that use of this product will prevent injury.

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Medical Specialties Inc.